

LED light & holder



Dziękujemy za wybranie przedniej lampy LED Trionic o mocy 200 lumenów.

W tej instrukcji pokazano, jak zamontować oświetlenie LED w Walkerze (lewa kolumna poniżej), a także jak zamontować je w Velopie (prawa kolumna poniżej).

Na zdjęciu po lewej stronie **A** przedstawia elementy mocujące uchwyt do Walkera (2 śruby + 2 podkładki). **B** przedstawia elementy mocujące uchwyt do Velopeda (1 śruba + 1 nakrętka radełkowana + 1 nakrętka).

Mocowanie uchwytu na światło


Instrukcje dotyczące chodzika



A1. Spośród dwóch dołączonych gumowych pasków wybierz ten, który ma 10-stopniowy kąt mocowania światła.



Instrukcje dotyczące Veloped

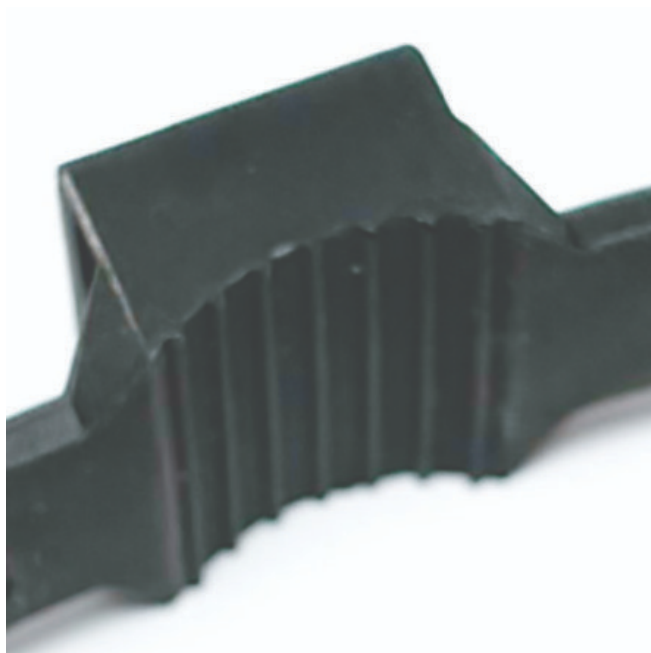

A1. Spośród dwóch dołączonych gumowych pasków wybierz ten, który ma sekcję mocowania światła pod kątem 0 stopni.



Instrukcja użytkowania

Pakiet zawiera

Jak ładować baterię

Ostrożność

1. Find the charging place at the bottom of the light.
2. Lift the rubber cover to access the charging port.
3. Plug the USB-cable into your USB-charger.
4. Plug the charging cable into the light's charging port.
5. The indicator shows red during charging.

Battery Performance Text
The charging indicator turns green when fully charged.

Attention: Use the original supplied USB-cable to charge the battery. Unplug the charging cable once the battery is fully charged.